




# 1

---


**Hi, do you know how to inject insulin?  
Ok, let me show you - it's quite easy!**

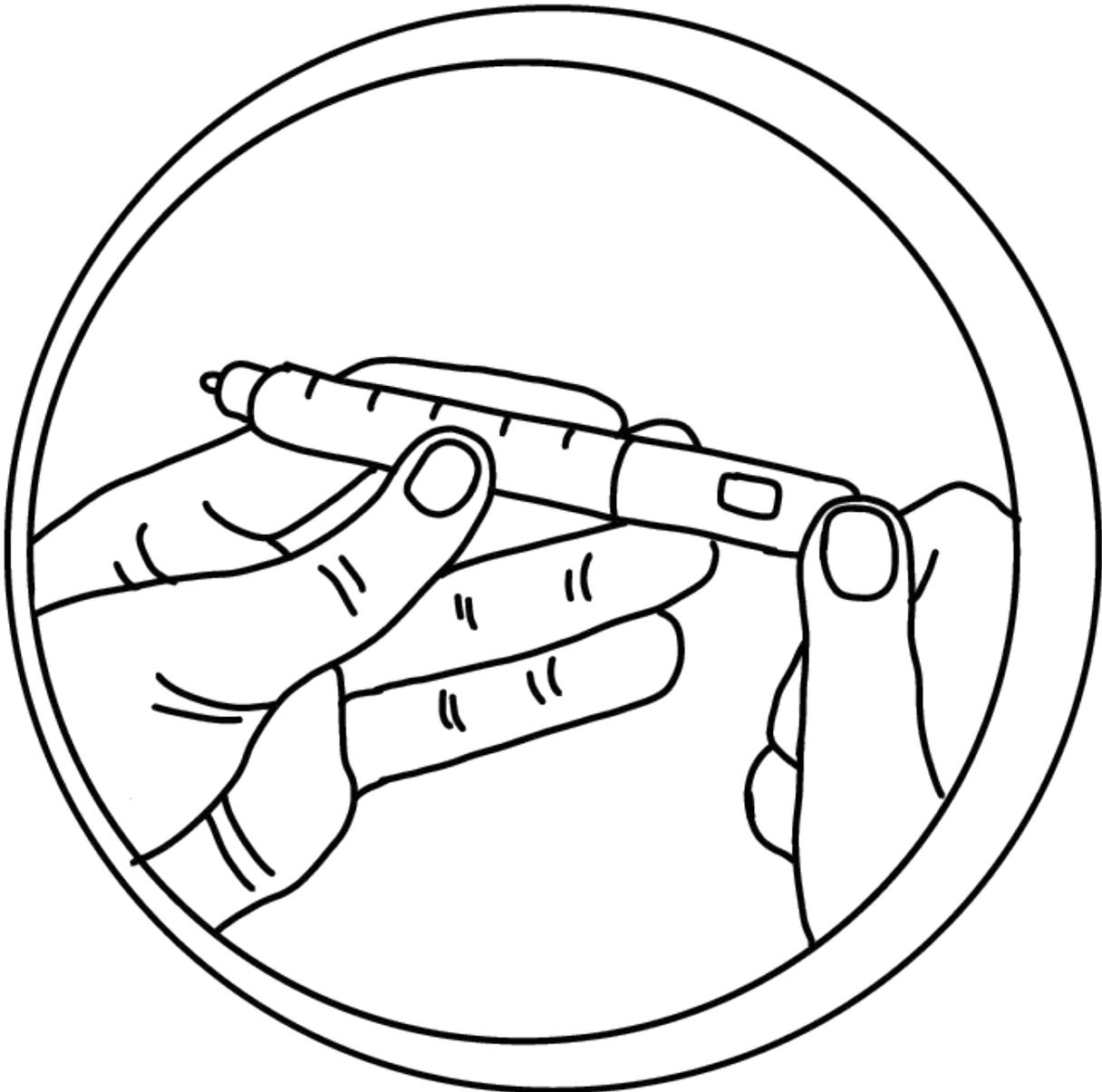

# 2

---

**This is my pen, it looks like an ordinary pen, but instead of ink it has insulin inside.**

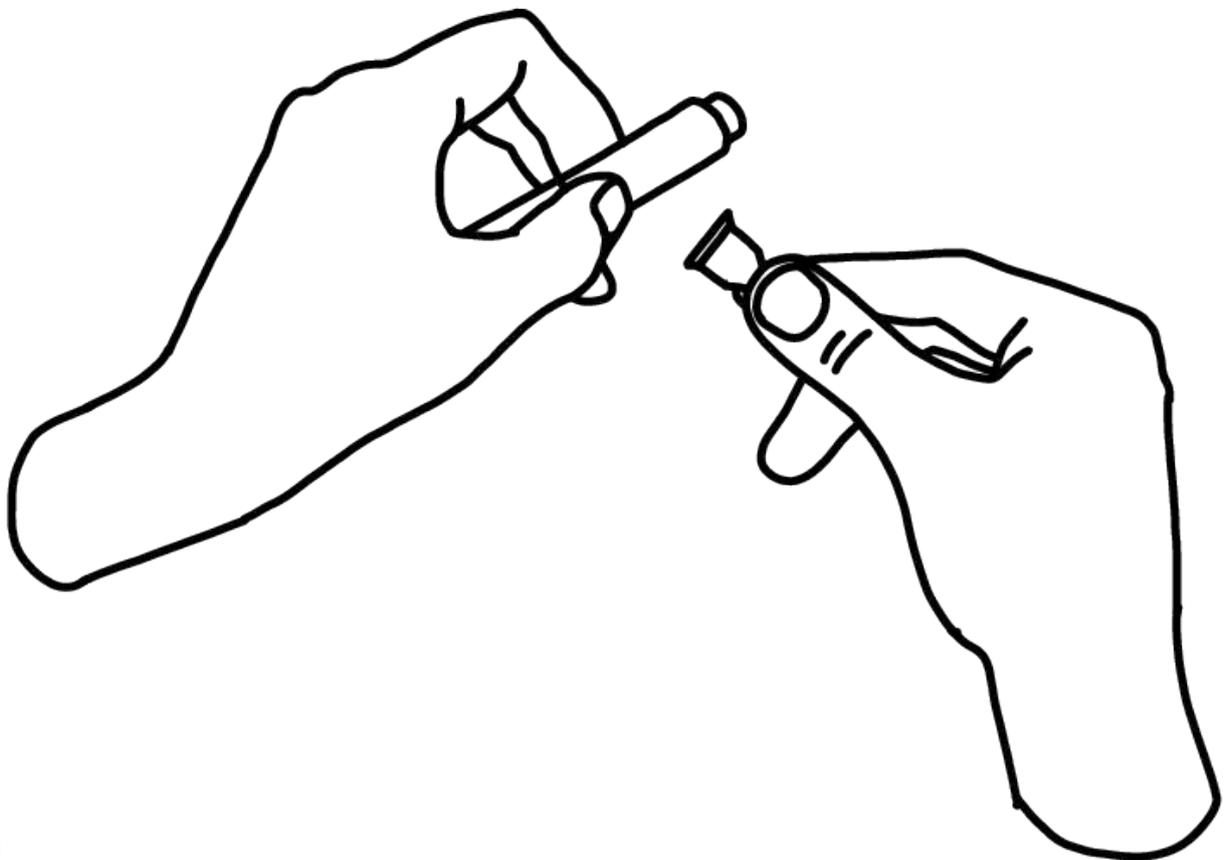

**3**

---


**First, we have to put a new pen needle into the pen. Remember to change the needle every time you inject insulin.**



**4**

---


**Now, prime your pen by selecting 2 units and pressing the dose button until you see a drop of insulin at the needle tip. Air can get into the needle and insulin cartridge, so priming helps make sure you get the correct dose!**

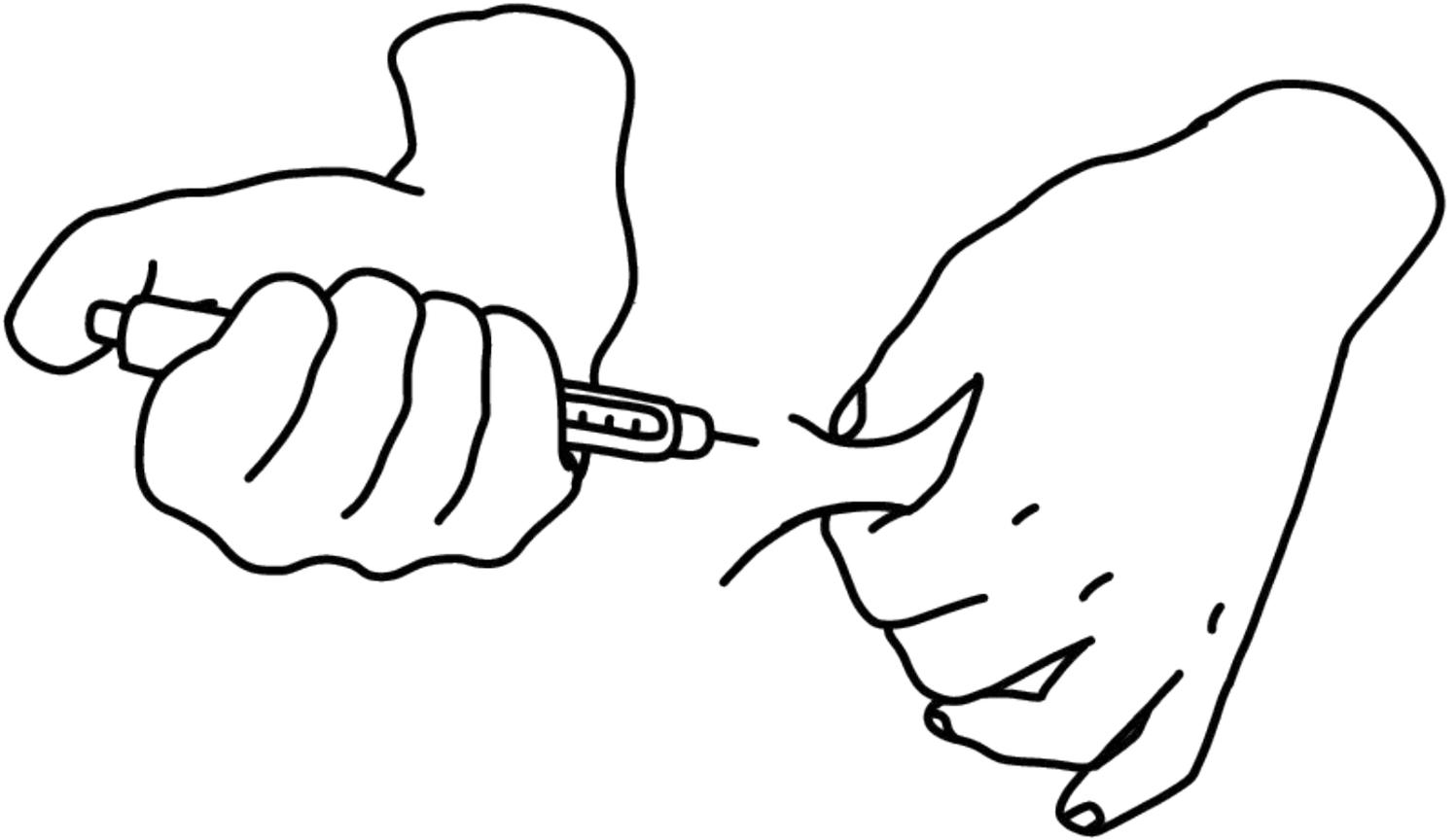

# 5

---

**Set your insulin dose and inject your insulin. Remember to push the dose button down completely and count slowly to 10, then remove the needle.**

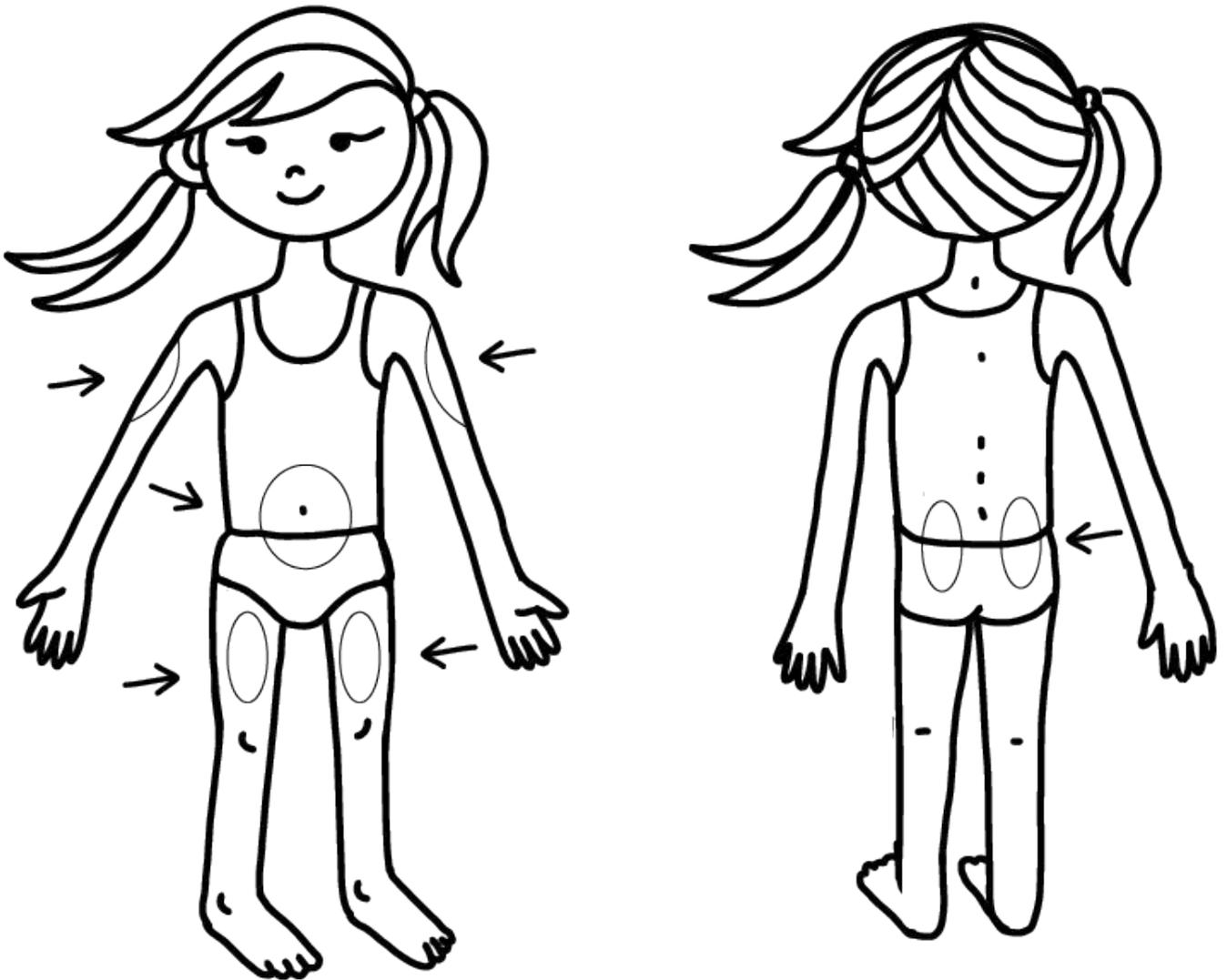

# 6

---

**You can inject into your belly, buttocks, and upper arms. It's important to switch your injection sites between those areas.**



**7**

---

**See, it was easy! See you soon!**