


IGŁY DO WSTRZYKIWACZY SĄ JEDNORAZOWE

Igły do wstrzykiwaczy kropelek są sterylne i jednorazowe. Nieuszkodzona plomba igły gwarantuje sterylność.

Pamiętaj! Igły do wstrzykiwaczy kropelek powinny być użyte tylko raz i wyrzucone!

PONOWNE UŻYCIE IGŁY MOŻE SPOWODOWAĆ:

- złamanie igły¹
- lipohipertrofię^{2,3}
- stępienie lub wygięcie końcówki, co z kolei może spowodować krwawienie, zasinienie lub blizny^{3,4}
- zwiększone ryzyko złamania igły w skórze¹
- zwiększyć ryzyko infekcji, ponieważ igła nie jest już sterylna⁴

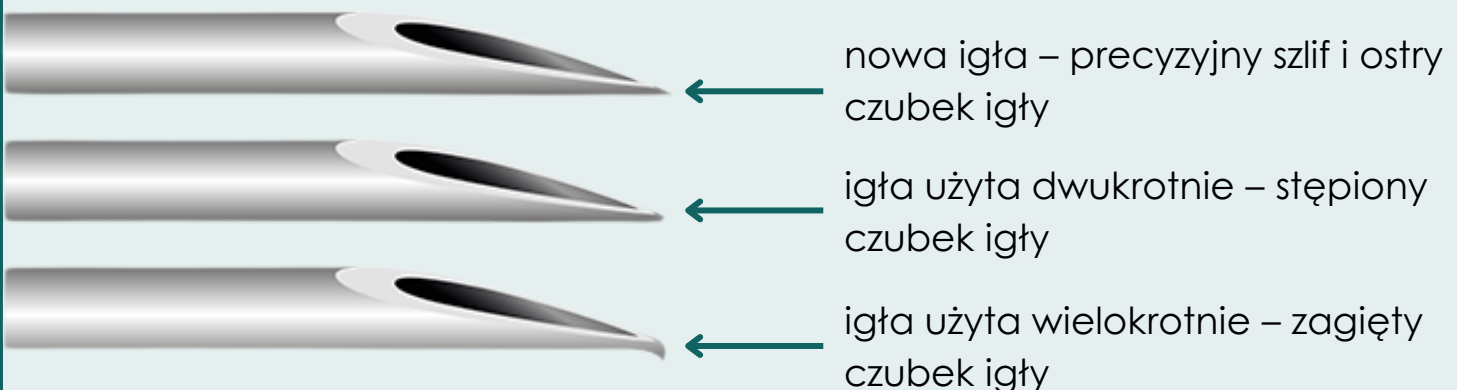

ŹRÓDŁA:

1. www.diabetes.co.uk; www.diabetes.co.uk/insulin/reusing-insulin-pen-needles.html
2. McNally P, Jowett N, Kurinczuk J, Peck R, Hearnshaw J. Lipohypertrophy and lipodystrophy complicating treatment with highly purified bovine and porcine insulin, *Postgrad. Med.* 64(1988) 850–853.
3. Frid AH, Kreugel G, Grassi G, Halimi S, Hicks D, Hirsch LJ, et al. New Insulin Delivery Recommendations. *Mayo Clin Proc.* 2016 Sep;91(9):1231-55
4. American Diabetes Association. Insulin administration. *Diabetes Care.* 2004;26(Suppl 1):S121–S124