EN Instructions for use of droplet® lanceting device

droplet® lanceting device is a medical device for multiple use with sterile, single-use lancets intended for capillary blood sampling by a lay person.

droplet® lanceting device is equipped with an optional transparent, Alternate Site Testing (AST) cap, which when installed on the device, allows blood collection from alternate sites.

This device allows to select 5 different depths of skin puncture.

- Remove the used lancet using the lancet ejector. Push the lancet ejector forward with the thumb and simultaneously pull out the sliding barrel to dispose of the used lancet in a safe way. Do not remove the used lancet from the device using your fingers.
- Prepare a new, sterile lancet.
- Carefully insert the lancet into the lanceting device and push it in until it comes to a complete stop.
- Put the adjustable cap back on.

CAUTION:

- The droplet® lanceting device is intended for a single user only.
- The lanceting device must NEVER be shared with any other person (including family members) due to risk of infection!
- For each puncture, a new, sterile lancet must be used.
- After the test is completed, the lancet should not be left in the device.
- Discard used lancet safely as per local regulations.
- Consult your healthcare professional before Alternate Site Testing (AST).
- Keep out reach of children without adult supervision.

The AST cap should not be used when:

- A low blood glucose level is suspected (hypoglycemia).
- There are rapid changes in blood glucose level (after a meal, after insulin dose or exercise).

- Remove the used lancet using the lancet ejector. Push the lancet ejector forward with the thumb and simultaneously pull out the sliding barrel to dispose of the used lancet in a safe way. Do not remove the used lancet from the device using your fingers.
- Prepare a new, sterile lancet.
- Carefully insert the lancet into the lanceting device and push it in until it comes to a complete stop.
- Put the adjustable cap back on.

KEY FEATURES:

- Sliding barrel
- Release button
- Lancet ejector
- Lancet holder
- Lancet
- Protective cap
- Clear AST cap
- Adjustable cap

BEFORE USE:

- Wash your hands thoroughly with soap and warm water to improve blood circulation in the fingers and then dry your hands.
- Remove the adjustable cap from the device by turning it anticlockwise.
- Prepare a new, sterile lancet.
- Carefully insert the lancet into the lanceting device and push it in until it comes to a complete stop.
- Twist off the lancet protective cap, being careful not to touch the needle.
- Twist off the adjustable cap from the device by turning it anticlockwise.
- After the puncture, hold the device until a drop of blood is formed under the transparent AST cap. If the drop is too small, gently massage the area around the puncture site until an adequate sample is obtained. Do not press the puncture site!
- Petrom the test.
- Carefully unscrew the AST cap from the device.
- Remove the used lancet using the lancet ejector. Push the lancet ejector forward with the thumb and simultaneously pull out the sliding barrel to dispose of the used lancet in a safe way. Do not remove the used lancet from the device using your fingers.
- Put the adjustable cap back on.

COLLECTING A BLOOD SAMPLE FROM ALTERNATE SITES (WITH THE AST CAP):

- Remove the adjustable cap from the device by turning it counterclockwise.
- Prepare a new, sterile lancet.
- Carefully insert the lancet into the lanceting device and push it in until it comes to a complete stop.
- Twist off the lancet protective cap, being careful not to touch the needle.
- Screw the transparent AST cap onto the device by turning it clockwise until it comes to a complete stop.
- Choose a puncture site different to the fingertip. It is important to avoid veins, moles, bones, tendons and wounds.

CLEANING AND DISINFECTION OF THE LANCING DEVICE:

- Never immerse the lanceting device in water or other liquids.
- The lanceting device should be cleaned and disinfected at least once a week.
- Use disinfecting wipes soaked with 70% isopropyl alcohol solution, available at pharmacies. Such wipes are safe for the device.
- Do not use bleach (sodium hypochlorite).
- The lanceting device maintains its parameters for three years, if cleaning and disinfection recommendations are followed.

PROCEDURE:

- Prepare a new disinfection wipe.
- Carefully wipe the whole lanceting device and the AST cap for at least 15 seconds.
- Allow the device to dry.

COLLECTING A BLOOD SAMPLE FROM THE FINGER TIP (WITH THE ADJUSTABLE CAP):

- Put the adjustable cap back on the lanceting device and twist it on clockwise until it comes to a complete stop.
- Set the puncture depth by rotating the adjustable cap dial. 1 - shallowest puncture 5 - deepest puncture
- Pull the sliding barrel at the end of device all the way back.
- Press the lanceting device against the fingertip and press the release button until the device clicks. (If there is no puncture, repeat the steps beginning with pulling the sliding barrel at the end of the device).
- After a drop of blood is obtained, perform the test.
- Carefully unscrew the adjustable cap from the device.

Press the lanceting device against the fingertip and press the release button until the device clicks. (If there is no puncture, repeat the steps beginning with the sliding barrel at the end of the device.)

To obtain a drop of blood, gently press the finger towards the puncture site. Do not press the puncture site!

After a drop of blood is obtained, perform the test. Carefully unscrew the adjustable cap from the device.

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