EN Instructions for use of **droplet®** lancing device

droplet® lancing device is a medical device for multiple use with sterile, single-use lancets intended for capillary blood sampling by a lay person.

droplet® lancing device is equipped with an optional transparent Alternate Site Testing (AST) cap, which when installed on the device, allows blood collection from alternate sites.

This device allows to select 5 different depths of skin puncture.

CAUTION:

- The droplet® lancing device is intended for a single user only.
- The lancing device must NEVER be shared with any other person (including family members) due to risk of infection!
- For each puncture, a new, sterile lancet must be used.
- After the test is completed, the lancet should not be left in the device.
- Discard used lancet safely as per local regulations.

The AST cap should not be used when:

- A low blood glucose level is suspected (hypoglycemia).
- There are rapid changes in blood glucose level (after a meal, after insulin dose or exercise).

There is asymptomatic hypoglycemia - the user does not have any symptoms of a low blood glucose level.

The results of alternate site testing do not agree with how the user (patient) feels.

The user is ill or stressed.

The user intends to drive a vehicle or operate machinery.

OTHER INFORMATION:

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the local competent authorities/government agencies, as applicable. If you have any health concerns, consult a healthcare professional. This device will work for two years of intensive use (12 uses per day) from the first use.

KEY FEATURES:

BEFORE USE:

- Wash your hands thoroughly with soap and warm water to improve blood circulation in the fingers and then dry your hands.
- Remove the adjustable cap from the device by turning it anticlockwise.

- Prepare a new, sterile lancet.
- Carefully insert the lancet into the lancing device and push it in until it comes to a complete stop.
- Twist off the lancet protective cap, being careful not to touch the needle.

COLLECTING A BLOOD SAMPLE FROM THE FINGER TIP (WITH THE ADJUSTABLE CAP):

- Put the adjustable cap back on the lancing device and twist it on clockwise until it comes to a complete stop.
- Set the puncture depth by rotating the adjustable cap dial. 1 - shallowest puncture 5 - deepest puncture
- Pull the sliding barrel at the end of device all the way back.

- Press the lancing device against the fingertip and press the release button until the device clicks. (If there is no puncture, repeat the steps beginning with the sliding barrel at the end of the device.)
- To obtain a drop of blood, gently press the finger towards the puncture site. Do not press the puncture site!
- After a drop of blood is obtained, perform the test.
- Carefully unscrew the adjustable cap from the device.

- Remove the used lancet using the lancet ejector. Push the lancet ejector forward with the thumb and simultaneously pull out the sliding barrel to dispose of the used lancet in a safe way. Do not remove the used lancet from the device using your fingers.
- Put the adjustable cap back on.

COLLECTING A BLOOD SAMPLE FROM ALTERNATE SITES (WITH THE AST CAP):

- Remove the adjustable cap from the device by turning it counterclockwise.
- Prepare a new, sterile lancet.
- Carefully insert the lancet into the lancing device and push it in until it comes to a complete stop.

- Twist off the lancet protective cap, being careful not to touch the needle.
- Screw the transparent AST cap onto the device by turning it clockwise until it comes to a complete stop.
- Choose a puncture site different to the fingertip. It is important to avoid veins, moles, bones, tendons and wounds.

- Press the sliding barrel at the end of device all the way back.
- Press the lancing device firmly against the chosen site and press the release button until the device clicks. (If there is no puncture, repeat the steps beginning with pulling the sliding barrel at the end of the device).

- After the puncture, hold the device until a drop of blood is formed under the transparent AST cap. If the drop is too small, gently massage the area around the puncture site until an adequate sample is obtained. Do not press the puncture site!
- Perform the test.
- Carefully unscrew the AST cap from the device.
- Remove the used lancet using the lancet ejector. Push the lancet ejector forward with the thumb and simultaneously pull out the sliding barrel to dispose of the used lancet in a safe way. Do not remove the used lancet from the device using your fingers.
- Put the adjustable cap back on.

CLEANING AND DISINFECTION OF THE LANCING DEVICE:

Cleaning and Disinfection

Clean and disinfect your lancing device once a week.

Use only Super Sani-Cloth germicidal wipes containing 55% isopropyl alcohol and 0.5% ammonium chloride. The wipes are proven safe to use with the lancing device.

Cleaning involves the removal of visible dirt and debris, but does not reduce the risk for transmission of infectious diseases.

Disinfecting (if performed properly) reduces the risk of transmitting infectious diseases.

Warning: If your lancing device is to be operated by a second person who is providing testing assistance to you, the lancing device should be disinfected prior to use by the second person.

Super Sani-Cloth germicidal wipes are available for purchase online. **Health Canada advisory:** Blood lancing devices for personal use may transmit blood-borne viruses if used on more than one person.

Signs of Deterioration:

Your lancing device has been tested for 156 cycles of the cleaning and disinfection procedure (one cycle per week for 3 years). The lancing device has been demonstrated to withstand 3 years of cleaning and disinfection without damage. However, if your device malfunctions for any reason, contact Customer Service at 1-877-660-1900 to troubleshoot and provide a solution.

Supplies needed for cleaning and disinfection:

- Super Sani-Cloth germicidal wipes
- Timing device

Cleaning Your Lancing Device

- Clean the lancing device with Super Sani-Cloth germicidal wipes until all soil is removed
- Allow to air dry

Disinfecting Your Lancing Device

- Before disinfecting, clean your lancing device as described in Cleaning Your Lancing Device.
- Remove the endcap and eject the lancet, if present.
- Get a new Super Sani-Cloth germicidal wipe.
- Set timer for 2 minutes and start timing.
- Wipe the complete lancing device continuously. You must wipe all surfaces and keep them wet for 2 minutes.
- Allow lancing device to air dry.

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